



**catalyst**

*Accelerating Pharmacy Practice*

## **PROGRAMMING HHP 4600R SCANNERS FOR RMS**

*VERSION 2.1*

*MODIFIED DECEMBER 1, 2008*



**To program an HHP 4600r Scanner for Catalyst RMS**

**Note:** Include a “|” as a prefix and Suffix for all scans.

1. Restore the standard product default settings



STANDARD PRODUCT DEFAULT SETTINGS

2. Program the imager for the USB PC keyboard



USB KEYBOARD (PC)

3. Clear all prefixes



REMOVES ALL PREFIXES

4. Add prefix



BEGINS PROGRAMMING SEQUENCE FOR ADDING A PREFIX

5. Enter the symbology type



9



9

CONTINUED ON NEXT PAGE

CONTINUED FROM PREVIOUS PAGE

6. Enter the hex value for the prefix/suffix (“|”)



7



C

7. Save



SAVE

8. Clear all suffixes



REMOVE ALL SUFFIXES

9. Add suffix



BEGIN PROGRAMMING SEQUENCE FOR ADDING A SUFFIX

10. Enter the symbology type



9



9

CONTINUED ON NEXT PAGE

CONTINUED FROM PREVIOUS PAGE

11. Enter the hex value for the prefix/suffix (“|”)



7



C

12. Save



SAVE